



CARE INSTRUCTIONS

DENTO-DORM® TREATMENT TABLE MATTRESS

To keep your **DENTO-DORM®** Treatment Table Mattress functional for many years, we would like to give you some tips on care and cleaning:

Your **DENTO-DORM®** Treatment Table Mattress consists of a rubber-containing cover with a foam core. The surface is resistant to pokes, scratches and solvents and is washable. It can be disinfected with standard surface disinfectants (not hand disinfectants).

For daily cleaning, we recommend wiping with a damp cloth and subsequently disinfecting with a conventional surface disinfectant. The **DENTO-DORM®** Treatment Table Mattress is reversible. In order to keep the surface of the cover shining, we recommend you wiping it from time to time with an oily cloth (e.g. olive oil).

When using anti-parasitic spot-on preparations, we recommend covering the Treatment Table Mattress. It can cause reactions with surface, which looks unattractive (white spots) but does not impair the function of the material.

We wish you pleasant work with your **DENTO-DORM®** Treatment Table Mattress.

The **DENTO-DORM®** Team.