



CARE INSTRUCTIONS

DENTO-DORM® POSITIONING MATTRESS

To keep your **DENTO-DORM®** Positioning Mattress functional for many years, we would like to give you some tips on care and cleaning:

Your **DENTO-DORM®** Positioning Mattress is made of polyethylene with a thermally sealed surface. The material is more robust than conventional foam and is produced without solvents. The **DENTO-DORM®** Positioning Mattress is absolutely waterproof.

For daily cleaning, we recommend wiping with a damp cloth or cleaning with running water, followed by spray disinfection with a conventional surface disinfectant. Afterwards, store the **DENTO-DORM®** Positioning Mattress in a dry place at an angle.

Adhering animal hair can be removed with the above-mentioned cleaning method. Please do not use a brush or adhesive rollers to avoid unnecessary strain on the sealed surface.

It is recommended to wrap each patient in a blanket or towel to increase the warming effect. For operations, the **DENTO-DORM®** Positioning Mattress should be covered with a sterile surgical drape. This will not affect the anti-slip surface of the pad

- **DENTO-DORM®** stabilises the body temperature of the patient during anaesthesia by means of countless small air cushions.
- **DENTO-DORM®** prevents patients from slipping even when the treatment table is inclined (4 padding wedges made of the same material are also supplied).
- **DENTO-DORM®** supports the positioning of the patient during dental radiography.



We wish you pleasant work with your **DENTO-DORM®** Positioning Mattress.

The **DENTO-DORM®** Team.